

Clearway

Foreign body removal

How to use the Clearway

- 1** Remove Clearway from package, attach to the vacuum supply using standard suction tubing (not supplied).
- 2** Manoeuvre the Clearway Suction Cup until it comes into contact with the embedded object, once contact has been established, create suction by placing your thumb over the Clearway suction control.
- 3** Whilst maintaining a seal, carefully pull out the Clearway to remove the object.
- 4** Please discard the Clearway Suction handle after use.

- Removes difficult foreign objects from ear and nose
- Ideal for A&E/ER and MIU
- Less traumatic for patient and family
- Unique system
- Can remove the need for ENT surgeon intervention or Theatre time



DTR Medical®