**INSTRUCTIONS FOR USE**

Frazier Suction Handles and bending for user preference  
**Product code(s):** FSH and FSF Product Groups

**READ ALL INSTRUCTIONS BEFORE USE!**  
**Single-use device. Do not resterilise**

**PRODUCT DESCRIPTION**

The Frazier Suction Handles are registered as a single-use sterile device. See packaging and labelling for product dimensions. The Frazier Suction Handles are used for a number of procedures in conjunction with a suction device and tubing. Each suction handle is furnished with a metal stylette. The device is double packed in a sterile sealed pouch.

**INTENDED USE**

The instrument is used with a suction device and tubing for various procedures in most areas of surgery. The suction handle can be bent to suit each surgeon's preference.


**WARNINGS AND PRECAUTIONS**

- Do not over bend or try to re-shape the device in any way other than what is stated below as this could cause stress fractures in the device and may render the device unusable.
- Proper surgical procedures and techniques are the responsibility of the medical profession. These instructions are for informational purposes only. Each surgeon must evaluate the appropriate use of this device, case by case, based on medical training, experience and the type of surgical procedure employed.


**USER INSTRUCTIONS**

Below are visual aids and instructions for a trained medical professional, if they wish to bend the Frazier Handle.

1.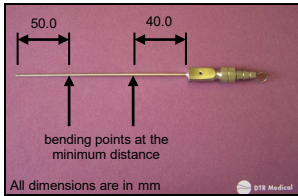


2.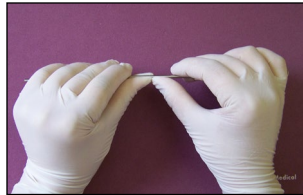


The recommended points at which to bend the device from each end. Ensure the suction control is in the favoured orientation.

Place both thumbs at the point of which requires the bend, and place index fingers firmly gripping the shaft for stability.

3.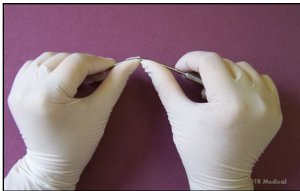

4.

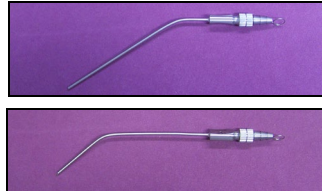Apply firm force by pushing from the thumbs up against the index fingers bending the tube but ensuring not to exceed an angle of 40 degrees

The top image shows the product bent at the recommended distance from the handle, while the bottom image shows the bend from the tip.

**Manufactured by:**  
DTR Medical Ltd.  
17 Clarion Close  
Enterprise Park  
Swansea  
SA6 8RF

Emergo Europe  
Westervoortsedijk 60  
6827 AT Arnhem  
The Netherlands  
(31)(0)70345-8570  
emergovigilance@ul.com

MedEnvoy Switzerland  
Gotthardstrasse 28  
6302 Zug  
Switzerland  
(41)415620142  
CH-REP@medenvoyglobal.com

Notified body:  
BSI  
Say Building  
John M. Keynesplein 9  
1066 EP Amsterdam  
The Netherlands

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